


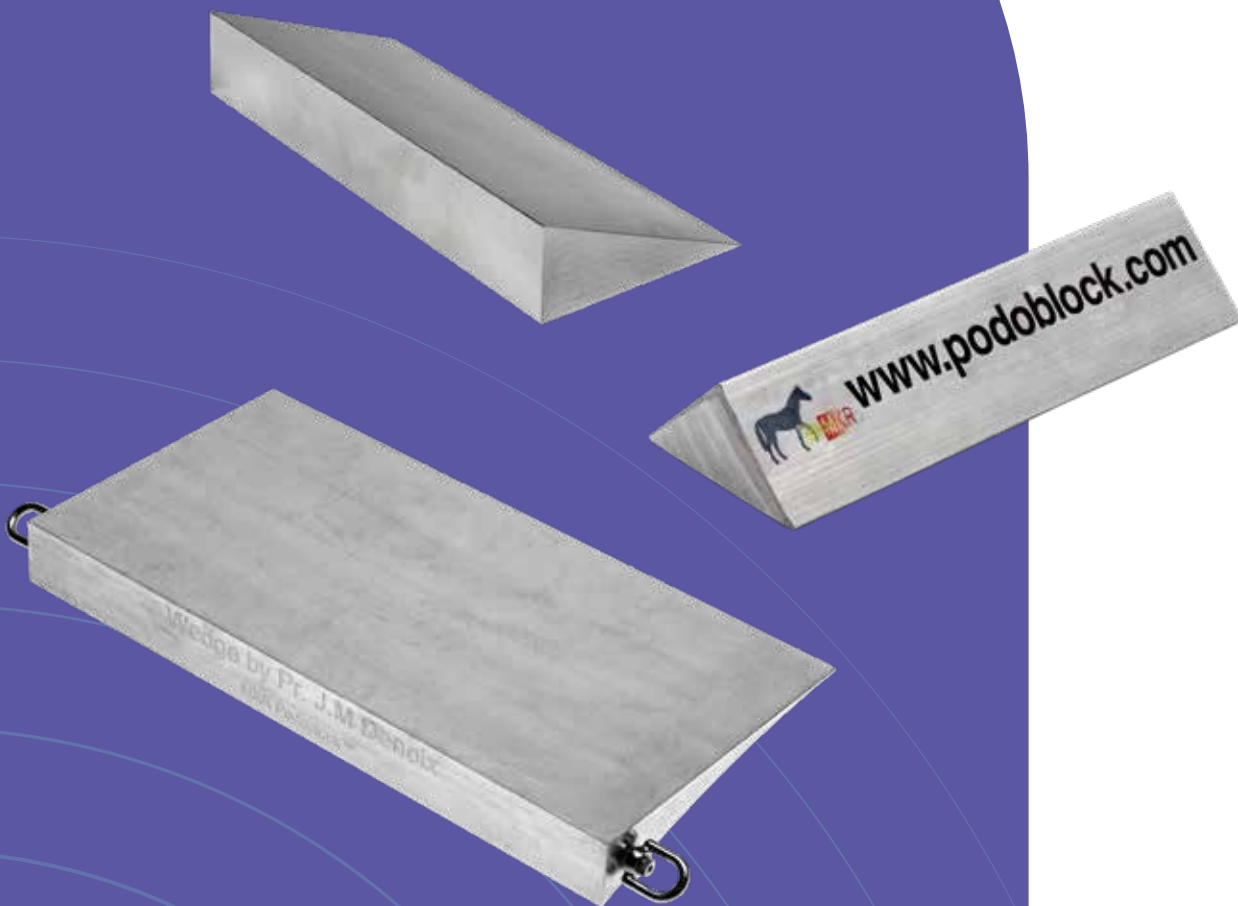

USER MANUAL

Aluminium Wedge

Australiëweg 10A

9407 TE Assen

The Netherlands

+31 50 8200 257

info@podoblock.com

podoblock.com



After sales correspondence

All correspondence relating to the after sales service of Podoblock B.V. to be addressed to:

NL Customers


Podoblock B.V.
Australiëweg 10A
9407 TE Assen
The Netherlands
Tel: +31 50 8200 257
Fax: +31 50 8200 212

Overseas customers

Contact your local distributor. In case of doubt contact Podoblock BV.
For further information visit www.podoblock.com

Trade marks

Podoblock™ is a trading name of Podoblock B.V.

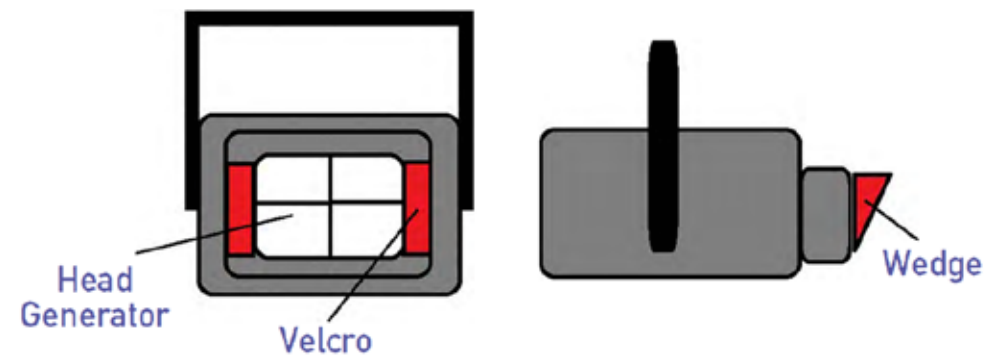

How to perform a good lateral radiographic view of the thoracolumbal spine of the adult horse




1. Use a powerful machine:
 - at least 60 to 80 KW
 - preferably use a DR system (for more sensitivity)
 - 105 to 120 kv and 90 to 180 mAs depending on the size of the horse
2. Have a legal and very effective radiation protection plan
3. Sedate the horse and place him perfectly square



4. Use a good grid (ratio 8 or 10) with a large cassette (30 x 40 cm)
5. Use and properly place the Aluminium Wedge
 - Place 2 lead markers on the back
 - The Aluminium Wedge is delivered with a contra-weight for ideal positioning and ease-of-use

Home-made aluminium wedge vs. Denoix Wedge

(same 580 kg horse, 115 kv, 1.1 m, 180 mAs)

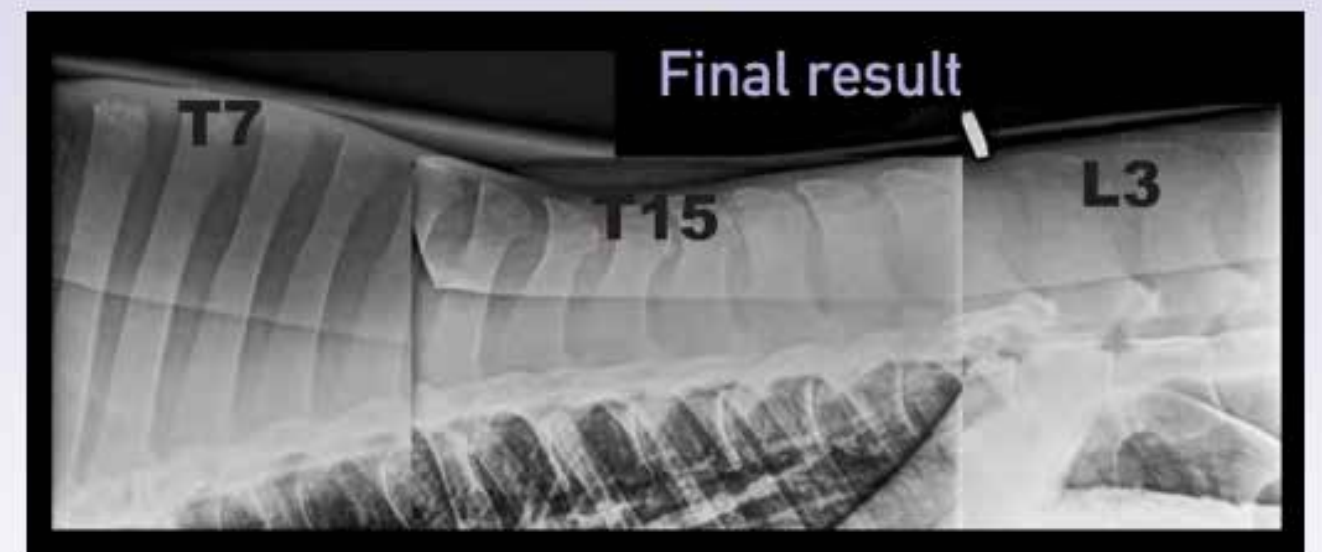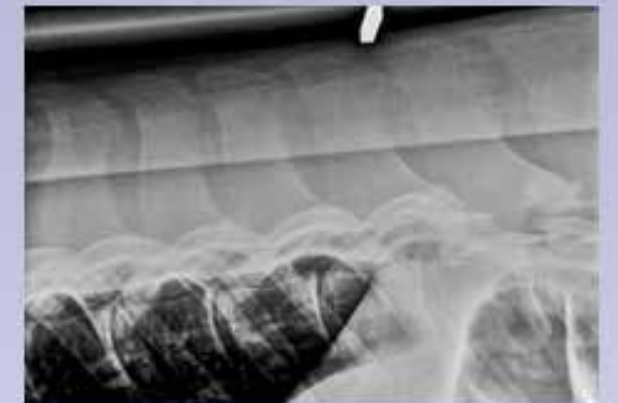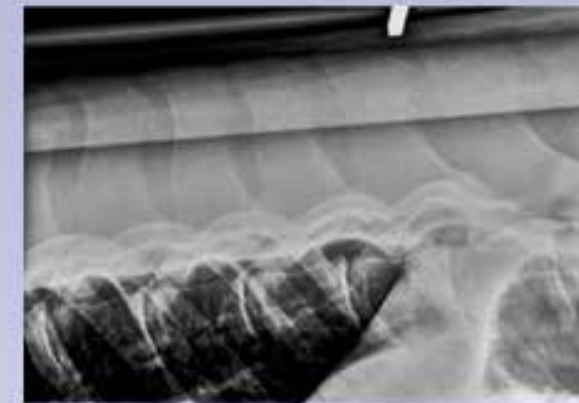

Courtesy of Dr. Jean-Marc Betsch, Clinique Vétérinaire Equine de Méheudin



Podoblock:

- Makes work easier
- Excellent service
- Safe and ergonomic working
- Global delivery
- Excellent quality products
- Customization
- Products easy to transport

Australiëweg 10A
9407 TE Assen
The Netherlands
+31 50 8200 257
info@podoblock.com
podoblock.com