



USER INSTRUCTION MANUAL  
**NewCart**

Verlengde Nijverheidsweg 10

9482 WT Tynaarlo

The Netherlands

+31 50 8200 257

info@podoblock.com

[podoblock.com](http://podoblock.com)



### After sales correspondence

All correspondence relating to the after sales service of Podoblock B.V. to be addressed to:

#### NL Customers

Podoblock B.V.  
Verlengde Nijverheidsweg 10  
9482 WT Tynaarlo  
The Netherlands  
Tel: +31 50 8200 257  
Fax: +31 50 8200 212

#### Overseas customers

Contact your local distributor. In case of doubt contact Podoblock BV.  
For further information visit [www.podoblock.com](http://www.podoblock.com)

#### Trade marks

Podoblock™ is a trading name of Podoblock B.V.

## 1.0 Preliminary information

### WARNING

Read this preliminary information carefully and note ALL of the warnings, cautions and safety notes contained within this manual before using the NewCart. Keep this manual close at hand at all times for reference. The NewCart must only be used as detailed within this manual. Failure to do so could result in injury to patients or users, or damage to the equipment. Podoblock is not taking any responsibility for improper handling with the NewCart.

- 1.1 This manual should be readily accessible for reference at all times and be referred to for details of the NewCart.
- 1.2 Sections marked with this symbol provide guidance on possible actions that could lead to injury of the patient or user or damage the NewCart. Ensure all warnings in the manual are strictly complied with.
- 1.3 Section 3 (Operation) provides information detailing how to use the NewCart. It is strongly recommended that the user has read - and is familiar with - section 3 before using the NewCart.
- 1.4 The NewCart is available for every type of Ultrasound source.

**Never** use the NewCart without first carefully reading this manual. Correct handling may prevent damage to the product or harm the user. Misuse, abuse of the NewCart will not be covered by the given warranty, neither will warranty cover the damage because of accident or high impact.



30 cm

45 cm

75 cm

106 cm

## 2.0 Description



Durable, in height adjustable cart for all small and big machines or equipment. Accommodated with gas spring for variable, stageless height-adjustment and storage space in bottom reservoirs. Easy transport position with the pull-out handle, easy storage in vertical position.

The NewCart has the following specifications:

- Transport size: 42 x 62 x 30 cm
- Weight: 19 kg
- Plateau size (inside rim): 57 x 37 cm
- Height-adjustment from 30 to 106 cm
- Maximum load: 15 kg
- 4 swivel, lockable Ø 100 mm castor wheels

## 3.0 Operation

Check for any wear-and-tear or damage that could compromise safety before using your NewCart. For setup and use of the NewCart please follow point 3.1 to 3.5



### 3.1 Unpacking and first impression

The NewCart is supplied in a cardboard box. If the cardboard box is damaged during transport and expected to have influence on the quality of the product inside, please contact your supplier immediately. When unpacking the NewCart, do not use any sharp tools or materials, this could damage the product. When completely unpacked, first check on visible damage and check if all parts have been provided, as mentioned in section 2.0.

### 3.2 Placing the NewCart

Before positioning the NewCart, make sure that the ground surface is clean and dry. The procedure of setting up should take place on a level ground surface. Make sure that your NewCart is placed under the conditions mentioned, otherwise it may cause damages to material and/or user.



### 3.3 Using the NewCart

The NewCart is equipped with 4 braked wheels. Make sure the wheels are set in the correct inward position.



When moving the NewCart with the shelf up, put the weight on the bottom as much as possible. This will reduce the strain on the top shelf and thereby reducing the risk of damages.



The NewCart can be easily transported in a trolley position. Carefully, and slowly pull the handle out making sure that the back wheels are locked in position before moving off.



To lower the shelf down, push in the lever and simultaneously push down the shelf, which will ease into a lower position or close.



To adjust the stand to your height preference (see height options on page 4), simply push the lever in and pull the shelf up simultaneously.



To adjust the stand to your height preference (see height options on page 4), simply push the lever in and pull the shelf up simultaneously.



Before loading the NewCart, please note that **the maximum carrying weight is 15Kg.**

Make sure that your heavy load is placed above the support arm as much as possible and **not on the end of the tray.**



Please **do not** load your NewCart as illustrated in the image on the left. Doing so will make the plate unstable and could potentially cause damages. **Refer to previous page for correct placement.**





#### Podoblock:

- Makes work easier
- Excellent service
- Safe and ergonomic working
- Global delivery
- Excellent quality products
- Customization
- Products easy to transport

Verlengde Nijverheidsweg 10  
9482 WT Tynaarlo  
The Netherlands  
+31 50 8200 257  
info@podoblock.com  
[podoblock.com](https://www.podoblock.com)