



GLOBAL LEADER OF ACCESSORIES  
FOR EVERY VET PRACTICE



WORLDWIDE DELIVERY  
INCLUDING TRACK & TRACE NUMBER



EXCELLENT CUSTOMER SERVICE  
WE'RE ALWAYS THERE TO HELP YOU

Verlengde Nijverheidsweg 10

9482 WT Tynaarlo

The Netherlands

[info@podoblock.com](mailto:info@podoblock.com)

Tel 0031 50 8200 257

fax: 0031 50 8200 212

Chamber of commerce (KvK): 58189998

VAT Nr.: NL852918227B01



# NewCart

## Ultrasound Solutions

## User Instruction Manual



[www.podoblock.com](http://www.podoblock.com)

# After sales correspondence

All correspondence relating to the after sales service of Podoblock B.V.

to be addressed to:

## NL Customers

Podoblock B.V.

Verlengde Nijverheidsweg 10

9482 WT Tynaarlo

The Netherlands

Tel: +31 50 8200 257

Fax: +31 50 8200 212

## Overseas Customers


Contact your local distributor.

In case of doubt please contact Podoblock B.V.

For further information visit [www.podoblock.com](http://www.podoblock.com)

## Trademark

Podoblock™ is a trademark of Podoblock B.V.

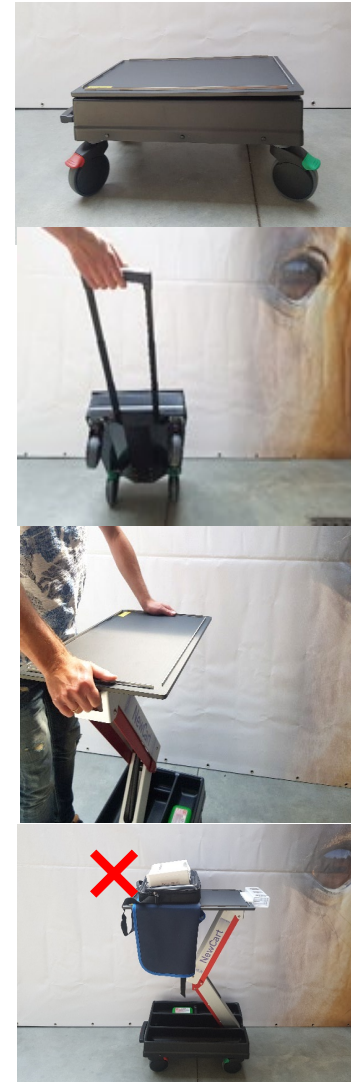


- When moving the NewCart with the shelf up, put the weight on the bottom as much as possible. This will reduce the strain on the stop shelf and thereby reducing the risk of damages.
- To lower the shelf down, push in the lever and simultaneously push down the shelf, which will ease into a lower position or close.



- Please **do not** load your NewCart as illustrated in the image on the left. Doing so will make the plate unstable and could potentially cause damages. **Refer to previous page for correct placement.**

#### FOLLOW THESE INSTRUCTIONS

- The NewCart is equipped with 4 braked wheels. Make sure the wheels are set in the correct inward position.
- The NewCart can be easily transported in a trolley position. Carefully, and slowly pull the handle out making sure that the back wheels are locked in position before moving off.
- To adjust the stand to your height preference (see height options on **page 2**), simply push the lever in and pull the shelf up simultaneously.
- Before loading the NewCart, please note that **the maximum carrying weight is 15Kg.**
- Make sure that your heavy load is placed above the support arm as much as possible and **not on the end of the tray.**

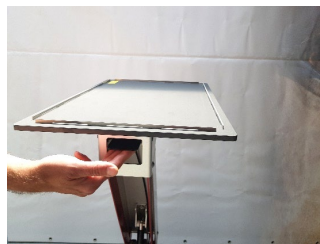

## 1. Preliminary Information



### Warning

Please read this preliminary information carefully and note **ALL** of the warnings, cautions and safety notes contained within these instructions for use before using the NewCart. Keep these **Instructions For Use (IFU)** close at hand at all times for reference. The NewCart **must only** be used as detailed within this “handbook”. Failure to do so could result in injury to patients or users, or (also) damage to the equipment. Podoblock does not take any responsibility for improper handling of the NewCart.

- 1.1** The instruction for use should always be readily accessible for reference and be referred to for details of the NewCart.
- 1.2** Sections marked with the symbol provide guidance to possible actions that could lead to injury of the patient or user, or damage to the NewCart. Ensure all warnings in the handbook are strictly complied with.
- 1.3** Section 3 **Operation** provides information detailing how to use the NewCart.
- 1.4** The NewCart is available for every type of Ultrasound source.

**Never** use the NewCart without first carefully reading this manual. Correct handling may prevent damage to the product or harm the user. Misuse, abuse of the NewCart will not be covered by the given warranty, neither will warranty cover the damage because of accident or high impact.

## 2. Description



Durable, in height adjustable cart for all small and big machines or equipment. Accommodated with gas spring for variable, stageless height-adjustment and storage space in bottom reservoirs. Easy transport position with the pull-out handle, easy storage in vertical position.



The NewCart has the following specifications:

- Transport size: 42 x 62 x 30 cm
- Weight: 19 kg
- Plateau size (inside rim): 57 x 37 cm
- Height-adjustment from 30 to 106 cm
- Maximum load: 15 kg
- 4 swivel, lockable  $\varnothing$  100 mm castor wheels

## 3. Operation



Check for any wear-and-tear or damage that could compromise safety before using your NewCart.

For setup and use of the NewCart please follow point 3.1 to 3.5

### 3.1 Unpacking and first impression

- The NewCart is supplied in a cardboard box. If the cardboard box is damaged during transport and expected to have influence on the quality of the product inside, please contact your supplier immediately. When unpacking the NewCart, do not use any sharp tools or materials, this could damage the product. When completely unpacked, first check on visible damage and check is all parts have been provided, as mentioned in section 2.2.

### 3.2 Placing the NewCart

- Before positioning the NewCart, make sure that the ground surface is clean and dry. The procedure of setting up should take place on a level ground surface. Make sure that your NewCart is placed under the conditions mentioned, otherwise it may cause damages to material and/or user.