




USER INSTRUCTION MANUAL

SCANdroid

Verlengde Nijverheidsweg 10
9482 WT Tynaarlo
The Netherlands
+31 50 8200 257
info@podoblock.com
podoblock.com

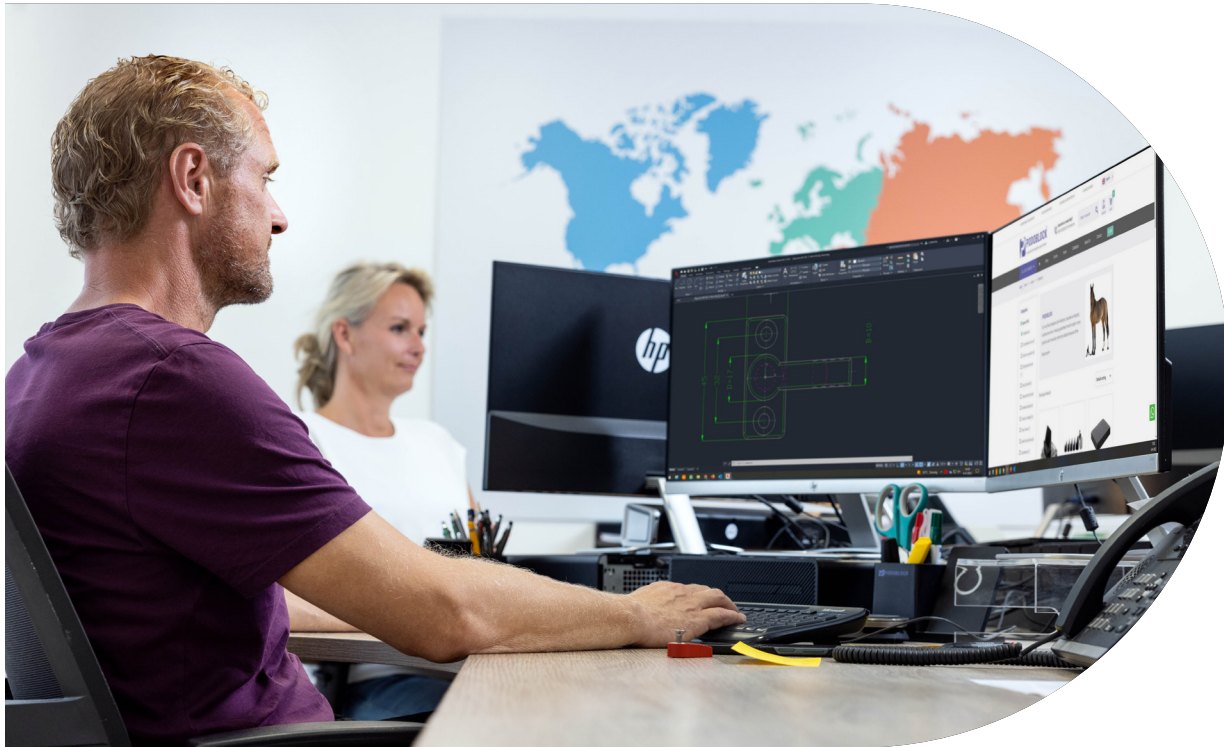

After sales correspondence

All correspondence relating to the after sales service of Podoblock B.V. to be addressed to:

NL Customers

Podoblock B.V.
Verlengde Nijverheidsweg 10
9482 WT Tynaarlo
The Netherlands
Tel: +31 50 8200 257
Fax: +31 50 8200 212

Overseas customers

Contact your local distributor. In case of doubt contact Podoblock BV.
For further information visit www.podoblock.com

Trade marks

Podoblock™ is a trading name of Podoblock B.V.



The CE marking affixed to the products certifies that it complies with the European Medical Devices Directive 93/42/EEC and related legislation number NL-CA002-2012-24231

1.0 Preliminary information

WARNING

Read this preliminary information carefully and note ALL of the warnings, cautions and safety notes contained within this manual before using the SCANdroid. Keep this manual close at hand at all times for reference. The SCANdroid must only be used as detailed within this manual. Failure to do so could result in injury to patients or users, or damage to the equipment. Podoblock is not taking any responsibility for improper handling with the SCANdroid.

1.1 This manual should be readily accessible for reference at all times and be referred to for details of the SCANdroid.

1.2 Sections marked with this symbol give guidance on possible actions that could lead to injury of the patient or user or damage the SCANdroid.
Ensure all warnings in the manual are strictly complied with.



1.3 Section 3 (Operation) provides information detailing how to use the different types of SCANdroid. It is strongly recommended that the user has read - and is familiar with - section 3 before using the SCANdroid.

The SCANdroid is only for use in the recommended ranges and therefore the warranty does not cover damage as a result of accident, abuse or high impact.



2.0 Description

The SCANdroid is available in one size and type.



Made of a compilation of loose components that are mounted together with aluminium rivets. The SCANdroid is supplied with separate or loose parts that are safely stored inside the SCANdroid. During transport it is not possible to lose this part if you have secured the seat with the Velcro strap.



The SCANdroid consists of the following parts:

- Round Ø40cm base with 5 swivel wheels, included 2 with locking
- Gas spring
- Seat with attached the height adjustment clamp
- Velcro strap at the outside to secure the seat during transport

3.0 Operation

Check for any wear-and-tear or damage that could compromise safety before use of your SCANdroid. For using the SCANdroid please follow point 3.1 to 3.6. If you need any extra info please visit www.podoblock.com, that will give you guides to use and also a short demonstration video.

3.1 Unpacking and first impression

The SCANdroid is supplied in a cardboard box. If the cardboard box is damaged during transport and expected to have influence on the quality of the product inside, please contact your supplier immediately. When unpacking the SCANdroid do not use any sharp tools or materials. This could damage your SCANdroid. When completely unpacked first check on visible damage and check the availability of all parts like mentioned in point 2.2.

3.2 Placing the SCANdroid

Before positioning the SCANdroid, make sure the ground surface is clean and dry. The SCANdroid must be placed on a level ground surface. **Beware that the SCANdroid is placed under the conditions mentioned, otherwise it may cause injuries to material and personnel users.**





3.3 First working height 23cm

At the outside is a grip handle that allows you to carry the SCANdroid. If the seat is on his transport height you can put the SCANdroid on his 5 swivel wheels and you have your first working height of 23cm.



3.4 Second working height 40cm

Untie the Velcro strap and secure it again at the outside. If you do not secure this the chance to fall over it is really great. Pull of the seat, grasp the gas spring and place the gas spring in the centre corner. After this place the seat on top of the gas spring. You now have your second working height of 40cm.



3.5 Third working height 50cm

Under the seat is a clamp for adjusting the height. To adjust the gas spring push the clamp down and go with your body weight of the seat, this will do come up the seat to his third working height 50cm.



3.6 Store the SCANdroid

After use first bring back the gas spring in to his shortest position. To do so, push the height adjusting clamp downwards and place your body weight on it. Remove the seat by pulling it off, remove the gas spring from the centre hole and store it inside the SCANdroid. Put the seat again on the base and secure it with the Velcro strap.

4 Cleaning and Service

The SCANdroid does not require servicing, however, a regular 4 monthly check for any wear-and-tear or physical damage that could impair its functionality is recommended.

This service check should include the following:

- Check the surface on all out side materials
- Check if the swivel wheels run free
- Check the gas spring on scrapes
- Check the grip handle
- Check on damages on the entire device
- If any damage is found, do not use the device and contact your local distributor

Cleaning with a disinfectant is necessary after every use.



Podoblock:

- Makes work easier
- Excellent service
- Safe and ergonomic working
- Global delivery
- Excellent quality products
- Customization
- Products easy to transport

Verlengde Nijverheidsweg 10
9482 WT Tynaarlo
The Netherlands
+31 50 8200 257
info@podoblock.com
podoblock.com